




# MatrixMag™ Home Treatment Kit Components

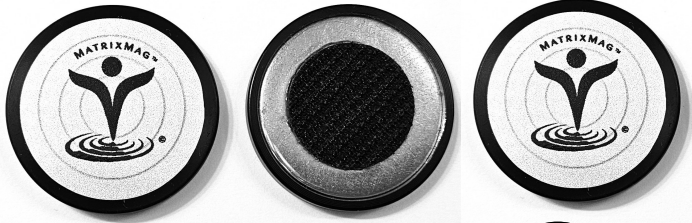


**ALTERNATING POLARITY CONCENTRIC MAGNET (APCM) WITH VELCRO BACKER:**  
Standard Kit: 3 Magnets - Plus Kit: 4 Magnets



**PATENTED APCM TECHNOLOGY**



**HANDY CARRYING CASE**

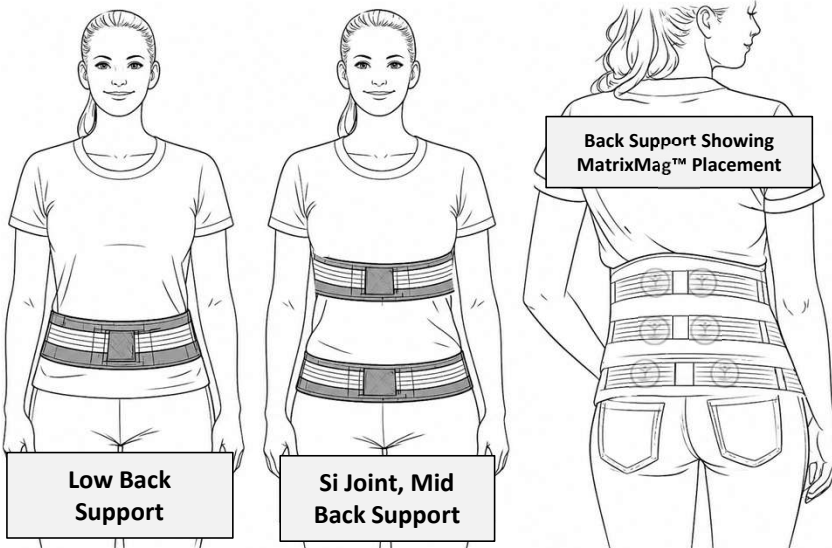

**Low Back Support**

**Si Joint, Mid Back Support**

**Back Support Showing MatrixMag™ Placement**



**Wide Use**  
Apply to Different Body Parts

- Finger  
- Wrist

- Foot  
- Knee




- Elbow

**BREATHABLE COHESIVE BANDAGE WRAP:**  
6 rolls 2 Inch x 5 Yards. **RE-ORDER ONLINE or MEDICAL SUPPLIER:** Self Adhesive Bandage Wrap or similar



**ADJUSTABLE KNEE SUPPORT:** Allows for MatrixMag placement.  
**RE-ORDER FROM WELLNESS SYSTEMS INC.**

- Waterproof
- Sweatproof

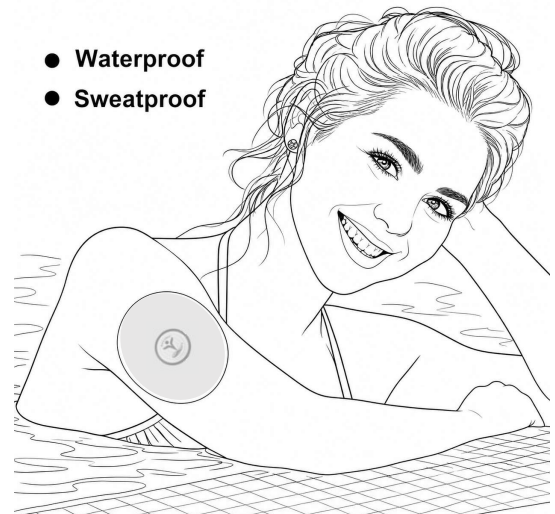

**WATERPROOF ADHESIVE PATCHES:**  
35 waterproof & breathable circular patches with non-adhesive center for MatrixMag placement  
**RE-ORDER ONLINE or MEDICAL SUPPLIER:** Adhesive Patches for CGM sensor.



# MatrixMag™ Home Treatment System

## Instruction for Use

**MATRIXMAG™**  
Advanced Magnetic Therapy

### Magnet Placement:

Magnets should be positioned so that the smooth (logo) side is facing the body. The magnets may be placed over a thin layer of clothing (back support, knee strap, bandage wrap) or next to the skin (patches).

### Duration & Frequency:

- Magnets can be in position for 1-4 hours (or longer if tolerated) per application, 3 to 4 times per day, with 30-to-60-minute breaks.
- Placement over center of chest, head or upper neck should initially be in place for a shorter duration (10-15 minutes) to monitor tolerance. Increase to maximum of 30 to 60 minutes application with equal-time breaks.


### Washing Instructions:

- Magnets: Wash in warm soapy water. Air dry.
- Back support, knee support: hand wash in warm water with mild soap. Air dry.

**Note:** Magnetic discs should not be positioned closer than 6 inches (15 cm) to an electronic medical device, such as a pacemaker or insulin pump.

### MatrixMag™ Universal Kit: Suggested Magnet Placement (Examples)

Placement of the MatrixMag magnets is based on the area of symptoms, or locations based on self-assessment or professional recommendations. Application with any of the fastening systems provided, is based on comfort and practicality.

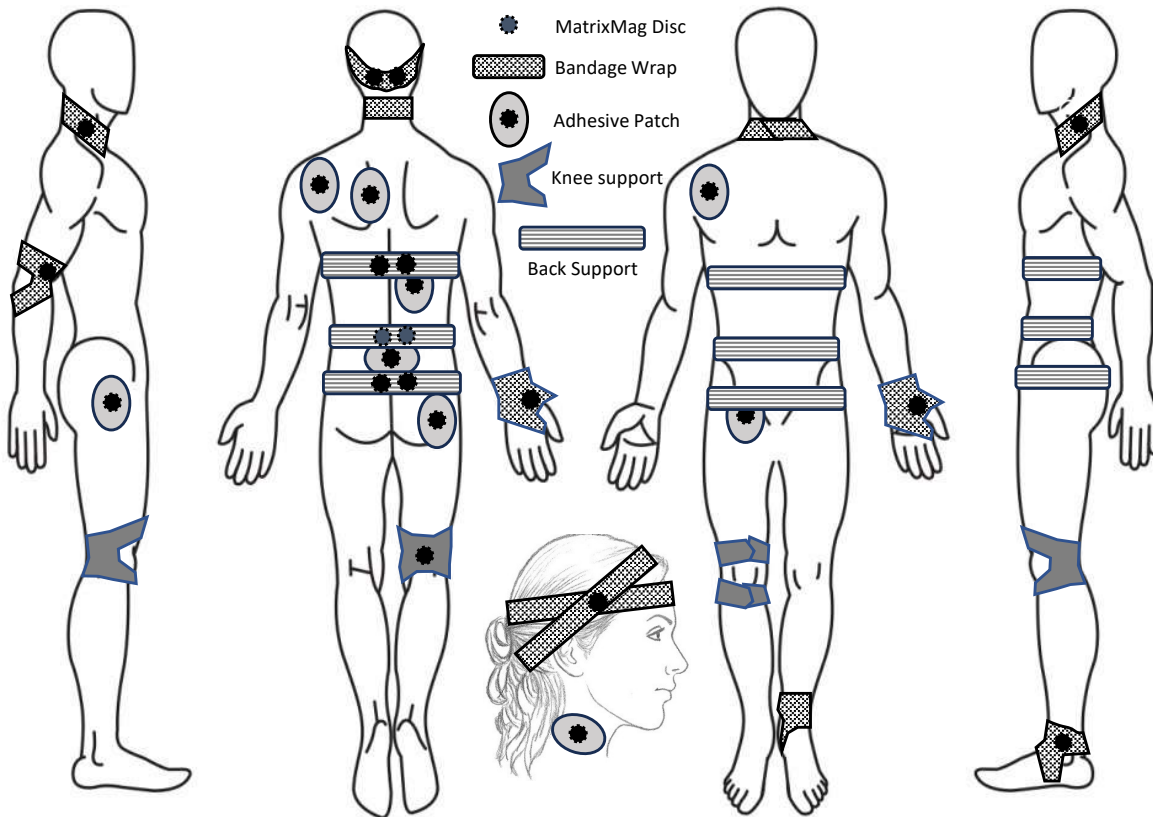

### MatrixMag™ Home Treatment System Limited Warranty\* and re-order information:

The MatrixMag™ magnets are guaranteed to be free of any manufacturing defects for 3 years. The back support and knee strap are guaranteed for 3 months from date of purchase. Bandage wrap and adhesive patches are excluded.

\*This warranty does not apply to improper use or damage sustained subsequent to purchase.